

# RIGHT FIRST TIME

Getting the best from DC Bead LUMI™ is simple – just follow these three easy steps:



### Get in the mix

Suspend DC Bead LUMI™ in pure contrast agent<sup>1</sup> at a ratio of at 1:10 and maintain good suspension throughout procedure.



### Give it space

Choose a 2.7/2.8Fr (ID >0.025"/0.650mm) catheter for initial cases. With experience, reduce to 2.4Fr (ID >0.021"/0.533mm) for 70-150µm if preferred.



### Go with the flow

Inject at 1ml/minute using a pulse motion (short, controlled pulses to match blood flow).

See more at [DCBeadLUMI.com](http://DCBeadLUMI.com)

DC Bead LUMI™ have loading with doxorubicin (for HCC) and irinotecan (for mCRC) within their CE-Mark approved indication statement.

For full instructions for use, warnings and important safety information relating to DC Bead LUMI™ please visit [www.dcbeadlumi.com/ifu](http://www.dcbeadlumi.com/ifu).

<sup>1</sup> Ask your BTG representative for details of recommended contrast agents.